
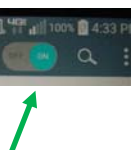



# ZOLL. X - 12 Lead Transmission

The Zoll X uses wireless internet to transmit 12 leads via the RescueNet. The “Zoll phones” currently issued to Rescue 1, 2 and 5 have Mobile Hotspots activated for this use. Zoll requires a secure network to transmit any patient data so networks cannot be added to the system without entering this configuration menu. Configurations are not available during patient monitoring.


Turn the phone on. Select and turn on the Mobile HotSpot app. The Active WiFi “Mobile Hotspot” connection can be verified by the icon at the top of the display screen on the X.

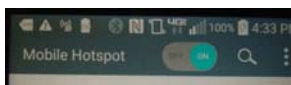


Wireless Connected

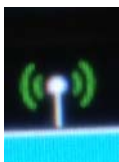

Wireless NOT connected:

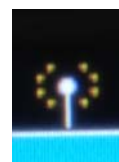

or

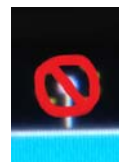


**Step 1: Select 12 Lead**




**Step 2: Acquire 12 Lead**



**Step 3: Select Envelope Icon to “Send”**

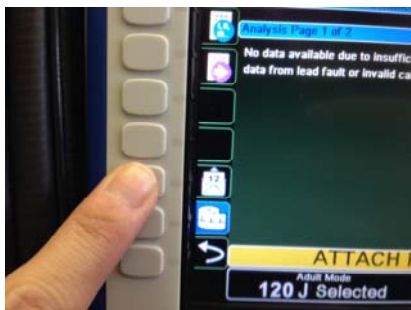

**Step 4: Select Destination from list and then Transmit**

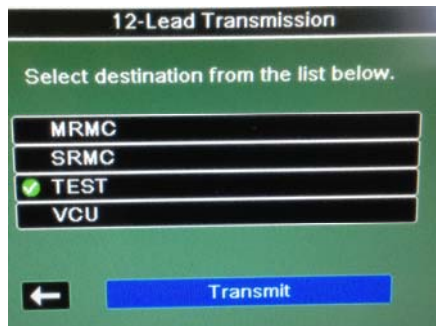

**Verify Transmission by receiving a green light on top of the X**