


HOW TO CLEAR ERROR CODE 45 FROM AN AUTOPULSE


1. HOLD DOWN THE ORANGE AND GREEN BUTTONS AT THE SAME TIME AND POWER ON THE AUTOPULSE. THIS WILL BRING YOU TO THE ADMIN MENU.



2. USE THE DOWN ARROW TO SCROLL DOWN TO "ROTATE SHAFT TO HOME" AND THEN HIT THE ENTER KEY ABOVE THE DOWN ARROW:

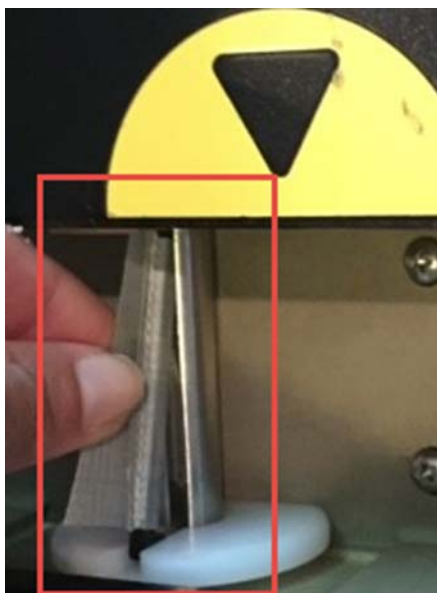

3. INSERT A BLACK CLIP FROM A USED LIFE BAND (BAND REMOVED) AND TURN THE DRIVE SHAFT UNTIL THE NUMBERS ON THE SCREEN DISPLAY "00000"

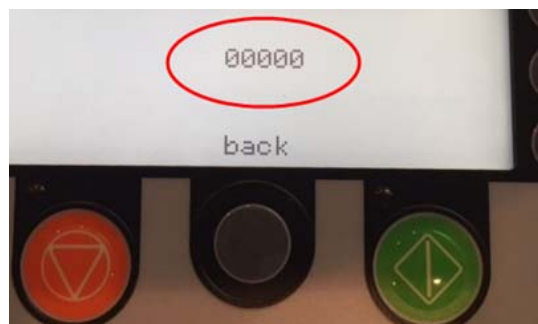

4. POWER OFF THE AUTOPULSE. REMOVE THE CLIP.
5. INSERT NEW LIFE BAND.